PORTUGUÊS

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HALLUX VALGUS PROTECTIONS with Activ Epithelium™

INDICATIONS

To provide protection and effective pain relief by reducing rubbing in cases of hallux valgus (bunions).

CONTRA-INDICATIONS

- Do not use on a skin lesion.
- In the event of an allergic reaction, or substantial irritation, the treatment should be stopped.

PRECAUTIONS FOR USE

- Protections should be applied to clean, dry skin.
- Remove a protection from the metallic envelope.
 - Remove the protective film covering the adhesive side.
 - Apply the adhesive surface to the skin, taking care that the area of the hallux valgus is well covered.
 - Apply light pressure to the protection to ensure that it has adhered to the foot.
 - Gently cover with socks, stockings or tights.
 - Keep the protection in place during the day.
 - Remove at night, place on the tray provided for this purpose and return to the metallic envelope until the next day.

For optimum results it is recommended that the protection is applied everyday.

WHAT IT IS USED FOR

- The effectiveness of these self-adhesive protections is due to Epithelium™ Activ gel which evenly distributes pressure and reduces painful rubbing.
- Very effective, and yet very thin, the protections may be worn in all types of shoes (formal, sports, house etc.) without introducing additional pressure.

These protections have been tested and approved for use by diabetics.

CLEANING

Protections may be washed with soap and water. Do not rub, allow to air-dry.

COMPOSITION: 85% silicone, 15% polyamide/elastane fabric.

The materials used have passed all the regulatory tests required by ISO standard 10993-1 relating to products that come into contact with the skin. If an incident occurs, stop using the product immediately and inform MILLET Innovation.

Millet Innovation cannot be held responsible for failure to use the product in accordance with the manufacturer's recommendations. French is the reference language. CE initial marking: 2006.

	Hand washable.		Do not iron or steam.		Do not dry clean.
	Do not bleach.		Do not tumble-dry.		See packaging.

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